

Introduction

Congratulations on your purchase of your new Raised Toilet Frame with Comfort Seat. Please ensure you read and understand these instructions in full before use. Please retain these for future reference.

Cefndy Raised Toilet frames are designed to enhance user comfort. The frame comes with integral arms and comfort toilet seat as standard. The feet are non-slip and non-marking for extra safety.

The frames comprise of a mild steel frame coated in a white or red antibacterial epoxy finish. All of the frames feature fully welded joints and clip on clip on clip off comfort toilet seats. All of these features are designed to aid product hygiene and reduce the possibility of bacterial entrapment.



T87
T870



T87R

Product Specifications

	T87 T87R	T870
Seat Height	470 – 620mm 18½ - 24½"	470 – 620mm 18½ - 24½"
Width Between Arms	475mm 19"	475mm 19"
Seat Width x Depth	390 x 420mm 15¼ x 16½"	390 x 420mm 15¼ x 16½"
Seat to top of arms	191mm 7½"	191mm 7½"
Footprint (Max)	490 x 555mm 19¼ x 21¾"	490 x 555mm 19¼ x 21¾"
Maximum User Weight	127kg 20st	127kg 20st

Pack Contents

- Raised Toilet Frame
- Comfort Seat
- Floor fixing Feet (T870 only)
- Non-Slip / Non-Marking extension legs (T87 & T87R only)
- User Instructions

Assembly Instructions

Please note:

- When unpacking carefully examine the product to ensure it is complete.
- Before use check for any damage. If damaged please contact your supplier and refrain from using the product.
- **Do not attempt to use if the product is damaged.**

Correct Use

Height Adjustment

- Remove the pin clip from each of the legs
- Select the required height of the frame by gently sliding each extension leg in or out.
- When desired height is found refit the pin clip into the appropriate hole in each extension leg
- Ensuring it goes fully through. Ensure that all legs are set to the same height

This product has been specifically designed to aid toileting requirements over an existing system.

Accessories

- A floor fixing foot kit is available – T174SET
- This allows you to replace the standard rubber tipped feet and replace them with feet that can be securely fixed to the floor.



To fit:

- Remove the pin clips from each of the 4 legs and remove the extension legs completely.
- Fit the 4 Floor Fixing Feet to the product and replace the pin clips – ensure the Fixing feet are set at the same height.
- Position the product exactly where required, and ensuring it is not moved, mark out the hole positions for each of the Floor Fixing Feet.
- Remove the product and using a drill bit of the type and size appropriate to the underlying floor material and whether plastic inserts are required. Drill each hole to the depth necessary dependent upon the use of plastic inserts and type of screws to be used. - If in doubt seek advice from professionals
- Using screws of the appropriate type and size, secure the product to the floor ensuring screws are fitted to all of the 12 holes in the Floor Fixing Feet.
- Remove any debris.

Cleaning

- Use mild detergent with warm water to wipe down framework on at least a weekly basis more frequently if required.
- Do not use an abrasive cleaner as this may damage the frame and invalidate your warranty.
- The toilet seat can be removed to aid cleaning.
- Remove legs for cleaning.

Use of power/jet washing & industrial washing machines

- If either of the above is used to clean equipment care must be taken since high-pressure water can be penetrative.
- Care should be taken that frames are not placed too close to high-pressure water jets.
- After cleaning ensure any residual water is removed from the framework as it could cause permanent rusting of the frame (leave frames open in a warm environment for several hours to ensure all internal moisture evaporates).
- We cannot offer advice on industrial washing machine

Maintenance Requirements

The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

Warning

- This product is intended for users up to a maximum of 127kg/20stone in weight.
- We recommend a full risk assessment is carried out before use. This should include consideration for the equipment's intended use, the capacity of the user to safely use the equipment, the environment in which the equipment is intended to be used and all other relevant considerations.
- It is advised that the frame is checked periodically for signs of wear and tear. Any problems please report to your supplier immediately.

Warranty

This product carries a 12 month warranty from the date of purchase against manufacturing defects. Misuse or damage is not covered. This warranty does not affect your statutory rights. As manufacturers we cannot specify expected product life as this is based on the level of usage.

Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to minimise the risk of infection.
- Materials used are essentially non-hazardous and can be disposed of appropriately consideration should be given to local re-cycling arrangements.

Contact Details

We have attempted to cover most eventualities, however if you have any specific queries, please contact:

Cefndy Healthcare
Cefndy Road
Rhyl
Denbighshire
LL18 2HG
Tel: 01824 706005
E-mail: cefndy.sales@denbighshire.gov.uk