

## USER GUIDE SLIDE SHEETS

The slide sheets are made in colours to indicate their size and have a contrasting loop for hanging.

The sheets are in the form of a tube with very low friction inner surfaces.

They are used for various positioning jobs like moving the patient up or down the bed or simply turning. Along with many others.

The two inner surfaces slide easily against one another avoiding any sheer force to the patient

### Instructions for use

First lay the sheet close to the heaviest parts of the body, hips/shoulders, then roll him/her onto the sheet. Now pull on the upper layer in the direction required. The sheet can be removed by gently easing out the bottom layer of the sheet.

### Care instructions

If the slide sheet needs washing please see below.



40° Wash



Low Tumble  
Dry



Do Not Dry  
Clean



Do Not Iron



Do Not Bleach

Pack contains:- One

SSWEL100  
200x100cm

Small  
70x66 cm

Medium  
120x70 cm

Extra Long  
200x70 cm

Large  
150x70 cm

Wide  
125x100 cm

Extra Wide/Long  
195x140cm