

WANCARE

ANTIBACTERIAL, ANTIFUNGUSIDAL AND VIRUSIDAL WASH GLOVES

%99.9
Effective

USAGE

It is used for general cleaning of the body area in inpatients and patients with limited mobility.
Apply glove the area that part of body which you want to clean.
Please use a separate gloves for each body area.
No need extra water, rinse and other cleaning products.
Non-rinse.

INSTRUCTION FOR MICROWAVE USE

1. Put the package into Microwave
2. Heat it for 20 seconds at 700 W
3. Check the temperature of wipes
4. Do not use higher than 35-45 C

PROPERTIES

99,9% effective.
Anti-fungicide.
Antiviral, antibacterial, antifungusidal.
Disinfectant wipes have been formulated for frequent use.
Kills 99.9% of tested bacteria in 30-60 seconds.
Nourishes and moisturizes skin with vitamin E, vitamin B and phospholipids.
Latex, lanolin and alcohol free.
Prevents bacteria with Chlorhexidine Digluconate

COMPOSITION

20 (%) Cons. Chlorhexidine Digluconate 2.0(%) Net Rate: 0.4 (%) w/w
Emulsifier, Preservative, Moisturizer, Surface Active Agent, Perfume, Water.

WARNINGS AND STORAGE CONDITIONS

Keep out of reach of children.
Keep away from heat, open flames and ignition sources.
Keep container tightly closed.
Store in cool and dry place.
For external use only.
Under normal conditions, shelf life is 2 years.
Use opened product within 3 months.
Avoid to contact with eyes. If in contact, wash the eyes with plenty of water.



BARCODE

8682560953273
8682560953280
8682560953297

REF

KAF G71
KAF G71-1
KAF G71-2

PACKAGE

8 PCS
10 PCS
12 PCS

KAFGRUP
SAĞLIK HİZMETLERİ

Atakent Mh. 221. Sk. No:3A Rota Office A Blok D.83
K.Çekmece/İSTANBUL
T:+90 212 471 4 200(pbx) F:+90 212 471 4 201
www.kafgrup.com / info@kafgrup.com